



Your treatment centre



To find out more about acoustic wave treatment, visit www.awt4you.com.

What does treatment involve?

Prior to treatment with acoustic waves, a layer of gel is applied between the skin and the handpiece. To stimulate the connective tissue, high-frequency acoustic waves are emitted. Treatment sessions may vary considerably depending on the indication, severity of symptoms and the area to be treated.

Frequency of treatment

6 – 10 sessions at intervals of 3 – 7 days

Duration of treatment

A session usually lasts 30 – 45 minutes.

Advantages

Non-invasive (the skin is not damaged), no rest period needed after treatment, all skin types can be treated.


Further information

References

- Adatto, M. et al.: Facial treatment with acoustic wave therapy (AWT) to improve facial skin texture, pores and wrinkles. *5CC*, 2016.
- Nassar, A. et al.: A randomized, controlled clinical study to investigate the safety and efficacy of acoustic wave therapy in body contouring. *Dermatologic Surgery*, 2015, 41(3), 366 – 370.
- Russe-Wifflingseder, K. et al.: Placebo controlled, prospectively randomized, double-blinded study for the investigation of the effectiveness and safety of the acoustic wave therapy (AWT) for cellulite treatment. *Journal of Cosmetic and Laser Therapy*, 2013, 15(3), 155 – 162.
- Adatto, M. et al.: Controlled, randomized study evaluating the effects of treating cellulite with AWT/EPAT. *Journal of Cosmetic and Laser Therapy*, 2010, 12(4), 176 – 182.
- Christ, C. et al.: Improvement in skin elasticity in the treatment of cellulite and connective tissue weakness by means of extracorporeal pulse activation therapy. *Aesthetic Surgery Journal*, 2008, 28(5), 538 – 544.
- Sattler, G. et al.: Pilot study: acoustic wave therapy (AWT) for cellulite. *Ästhetische Dermatologie*, 2008, 2, 16 – 25.

SM 30 189 E 0119/51H - Subject to change without notice.

No need to hide

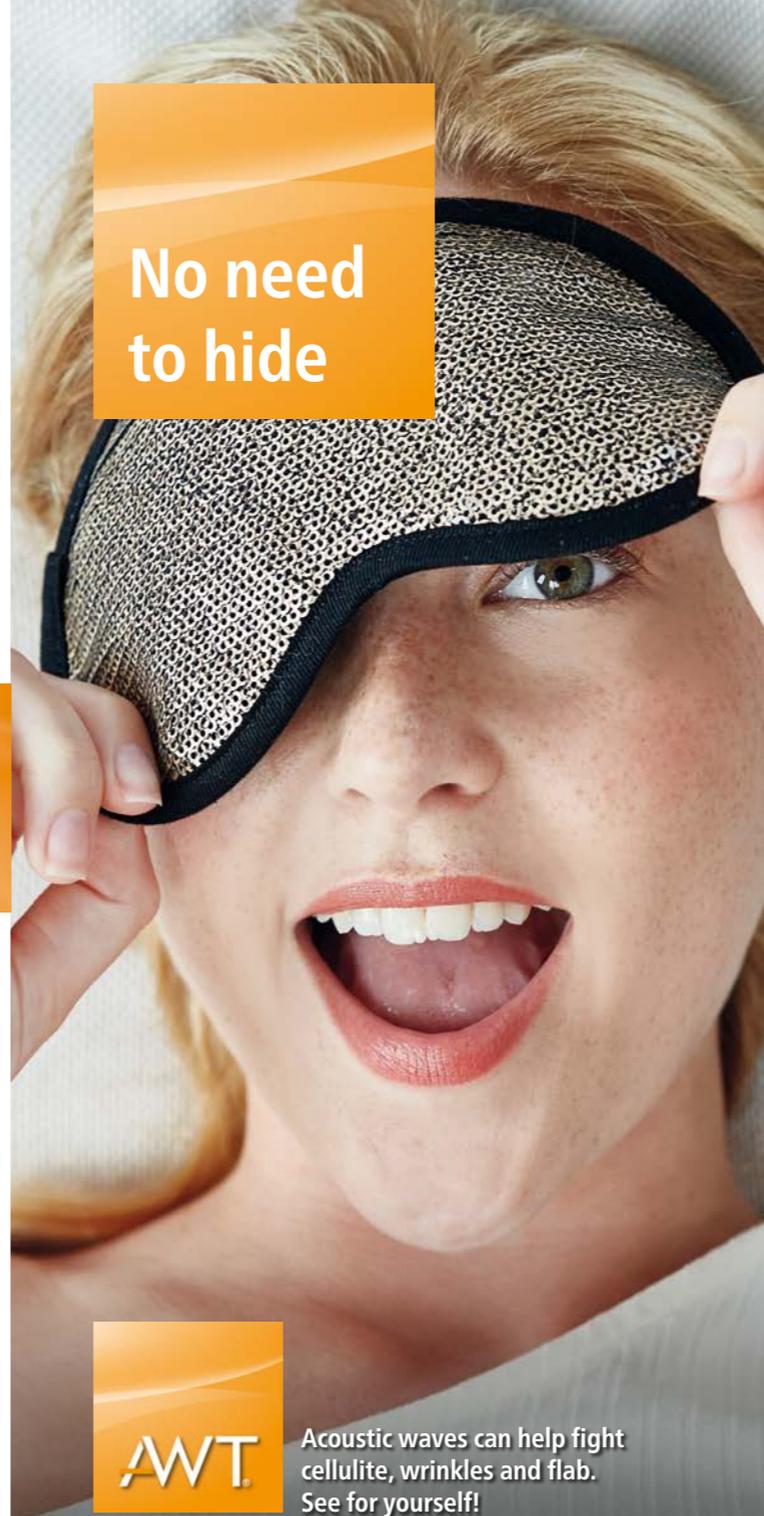

Acoustic waves can help fight cellulite, wrinkles and flab. See for yourself!

What is AWT®?

AWT® (Acoustic Wave Treatment) involves introducing acoustic waves into affected regions of the body. Medical applications for these acoustic waves go back to 1980; they have been successfully used in pain therapy and in treating a wide range of other conditions. Recent studies show that acoustic waves have beneficial biological effects that, in aesthetic treatment, may lead to activation of metabolic processes and stimulation of connective tissue. Therapy outcomes are generally excellent and benefits may last up to six months.

AWT® application





Cellulite

Cellulite, and the typical »orange peel« appearance of the skin associated with it, affects the majority of women. It results from an increase in stored fat, lymph and/or fibrotic tissue in the skin, and is present mainly in the buttocks, hips, thighs and upper arms. The enlarged fat cells and lymphatic deposits are pushed upwards through the connective tissue. This results in the typical »orange peel« dimpling of the skin surface.

- AWT® mechanically stimulates the tissue.
- AWT® improves the microcirculation.
- AWT® smooths the skin.

Body shaping

Our bodies are always trying to optimize their energy use as far as possible. If we take in more calories than are burned, the excess is stored in the form of fat reserves that we can fall back on as required. Part of this fat goes into storage in certain favoured regions of the body that accumulate excessive lipids. Acoustic waves can be used to treat the small to medium-sized fat deposits formed in this way.

- AWT® reduces the subcutaneous fat layer.
- AWT® helps to reduce circumference.

Skin tightening

The breakdown of the collagen fibres and elastin causes the skin to lose tension. This looseness of the skin is generally due to poor nutrition or unsuitable weight-loss diets, an unwanted side-effect of which is loss of muscle mass. The result is that certain regions of the body, including the buttocks, upper arms, abdomen and inner thighs, appear soft and flabby.

- AWT® increases skin elasticity.
- AWT® increases skin tightness.
- AWT® improves the microcirculation.

Wrinkles

Our skin begins to age from our mid-20s onwards. The connective tissue cannot store as much moisture as it used to, and elasticity decreases. The natural ageing process is exacerbated by external factors such as UV exposure. This results in the notorious nose/cheek wrinkles, forehead wrinkling, drooping jowls and fine lines around the nose, eyes and mouth.

- AWT® reduces wrinkle depth and width.
- AWT® reduces pore size.
- AWT® improves skin texture.